

# GUIDO AMMIRATA

www.ammirata.it

## CAVO PAZIENTE PER ECG

## ECG PATIENT CABLE

### CHE COSA È IL CAVO PAZIENTE? WHAT IS THE PATIENT CABLE?

Il cavo paziente serve a collegare le apparecchiature (elettrocardiografi, monitor, registratori holter ECG) agli elettrodi applicati al corpo del paziente.  
The patient cable is used to connect equipment (ECG, monitors, ECG holter registers) to the electrodes applied to the body of the patient.

### COME È FATTO IL CAVO PAZIENTE? HOW IS THE PATIENT CABLE MADE?



SPINOTTO PLUG	CORDONE CORD	PARTITORE DIVIDER	TERMINAZIONI TERMINAL ENDS	CONNETTORI CONNECTORS
La forma varia in base all'apparecchio al quale si connette	È la connessione elettrica tra lo spinotto e il partitore. Possono essere lunghi: - 200 cm e vengono usati per elettrocardiografia a riposo - 350 cm e si usano per ergometria	È una scatola pressofusa in cui entra il cordone e contiene una piccola scheda elettronica dalla quale escono le 10 terminazioni.	Sono 10: - 4 lunghe 100 cm per gli arti dette Periferiche - 6 lunghe 75 cm per il petto dette Precordiali	Può essere di 3 tipi diversi a secondo degli elettrodi utilizzati: A) a bottone (snap) B) a banana 4mm C) a clip (pinzetta)
<i>The shape varies according to the equipment to be connected to</i>	<i>It is the electrical connection between the plug and the divider. The lengths may be: - 200 cm and are used for ECG at rest - 350 cm and are used for ergometry</i>	<i>It is a die-cast box in which the cord enters and contains a small circuit board from which 10 terminal ends come out</i>	<i>Are 10: - 4, 100 cm each for the limbs, called peripheral - 6, 75 cm each for the chest, called precordial</i>	<i>They may be of 3 different types, divided according to the kinds of electrodes used: A) Button (snap) B) 4 mm banana shape C) Clip</i>

**CARDIETTE** - ELETTRONICA TRENTINA DXP1000, Microscan 2000, HR one 48 • All versions of AR600, 1200, 2100; Start 100P, Start 100H, Start 200, 200HV • DOC 101,103; Autoruler 12/1, 12/3; Excel 103, 106, 106SCP; Excel 112, 212; Daedalus 346; Serie Daedalus View • Every model of Micro, Microruler, Microtel • Hr One, Hr One48

**CARDIOLINE** all versions of AR600, 1200, 2100; El, El R, E3; ECGI00+, ECG200+, ECG100S, ECG200S • Click Holter • Delta 1, 3, 30, 30D, 60, 60D, lplus 3plus, 3/6plus • Elan, Elanl 100 • Delta 60plus, ElanStress • Delta 300, 360

**EDAN** SE-1, -100, -3, -3WS, -300, -301, -6, -600, -601 A-B-C, -12, -12 ex press, -1200, -1200 express, -1010 (not wireless), -1201, -1515 DPI 2, ST-1212; VE-100, -300, -1010

**EDAN-INS** ECG-1 A, -1B, -3A, -3A plus, -3B, -3B plus, -6A, -6B, -12A

**ESAOTE** Lifecard CF (3-ch x 3 leadwires); Lifecard CF 3-ch/7-leads: 7 DIN-to-snap leadwires for trunk cable (trunk cable not included) • P80, P80six, P80power, P8000, P8000power, P8000 tele, Bluecg, MyCardiopad, MyCardiopad XL, AT-10 plus, ATI 02 plus, Cardiovit M2015

**FUKUDA DENSHI** Autocardiner: FCP11, FCP2101A, FCP220A, FCP4101A FX101. MCP302 • Cardimox: FX2111, FCP2155, FCP7101, FX3010, FX4010, FX7102, FX7202, FX7302, FX7402, FX7542, FX8222, FX8322, FX8328 • Alpha 500, Alpha 600

**GIMA** Cardiopocket, Smart SE-1, -3, -6, -12 Express. Gimaplay 1 • Cardiogima 1M, 3M, 6M, 12,12M, Colour ECG. PC-ECG 33320, 33321; Acquisition box series MGY • ECG Holter ref. 35130

**HELLIGE** EK36, EK41, EK43, EK56, EK403, EK413, EK512, EK512 E, EK512, P. EK51, Corina • EK53

**HEWLETT PACKARD** PageWriter 100,200,300 (M1770A, MI 771 A, MI 772A)

**MINDRAY** Datascope Spectrum & Passport 2 • DECG-03A, BeneHeart R3, BeneHeart R12

**MORTARA** Eli 10,10 mobile, 150, 210, 250, 350, 150c, 250c 380 • Eli 00, 200. X-1 2, H-1 2, X-1 2+, H-1 2+, Surveyor II / X-1 2 Transmitter

**NIHON KHODEN** ECG-1150 cardiofax C, 1250 cardiofax S, 1350 cardiofax M, 1500/-1550 cardiofax V, 2350 cardiofax M, PEA-1110 • ECG-651, 9010, 9020, 9022, 9110, 9130, 9132 cardiofax Q, 9620 • Cardiofax 6151,6353, 8110, 8240, 8270, 8330, 8340, 8350, 8370, 8420, 8423, 8450, 8453, 8820, 8823, 8830 • Cardiofax Q

**PHILIPS HP** PageWriter 10, 10i, Pagewriter 1513,154B/C, 1514D, 1515A/B, 1517A • IntelliVue Telemetry • StressVU • TC20 • TC30-50-70 • C3i, Efcia ECG100

**PHYSIO CONTROL** Ready Link

**SCHILLER** AT-1, AT-2, AT-2plus, AT-2 light, AT-3, AM, AT-101, AT-104, AT-110 CS-6, CS-6/12, CS-12, AT-5, AT-6, AT-10, AT-10+, AT-60, AT-100


• MS-2007, AAS-2010, MS-2015 • AT-10plus, AT-102, AT-104PC, CS-100, CS-200, CS200 excellence, CS200 touch etc.

**SIEMENS** Cardiostat 1, 11, 31, 31S • Cardiostatl, 3T, 703, 710, 31, 31S; Cardirex 3T, 4, 6T, 62; Megacard Elema A/B, R, R/E; Cathcor, Cardina C, Miracard 1; Sirecard 440, 440S, 460; Mingodoc, Mingocard 3, 4, 7; Mingophon 3, 7, 310, 410, 420, 710, 720; Mingograph 4, 62, 81, 82, 740; Mingolog 7

**SPACELABS - DEL MAR REYNOLDS** Lifecard CF 3-ch: 7 DIN-to-snap leadwires for trunk cable (trunk cable not included) • Lifecard CF (3-ch x 3 leadwires); CardioExpress SL3, SL6, SLT 2 • Voyager V6, V12 • 90205, 90208, 92513 • Ultraview SL2400,92513

**WELCH ALLYN** Cardio Perfect • CP50,150,300 • Serie AT • SerieSP • SerieCT • Serie AT • SerieSP

**ZIMMER** Cardio 1, Cardio 3, Cardio 5, Cardiocompact

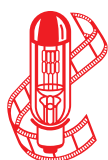

**GUIDO AMMIRATA S.R.L.**  
**MEDICAL LAMPS**

Via Marocco 13 - 20127 Milano (Italia)  
Tel. +39 02 2820646 - Fax +39 02 2894660  
info@ammirata.it - www.ammirata.it